THERA® L 40SE

Ball-bearing turbines
Turbine a cuscinetti a sfera
Turbine mit Kugellagern
Turbines à roulements à billes
Turbinas de cojinetes de bolas
Turbinas de rolamentos de bolas
Γωνιακή αερότορ με ρουλεμάν

REF 160.02.02

CE
0051

Operating instructions
Istruzioni per l'uso
Gebrauchsanleitung
Mode d'emploi
Modo de empleo
Instruções de uso
Οδηγίες χρήσεως

TKD

TeKne Dental

Fig. 1

Fig. 2

Fig. 3

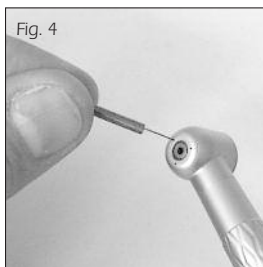
Fig. 4

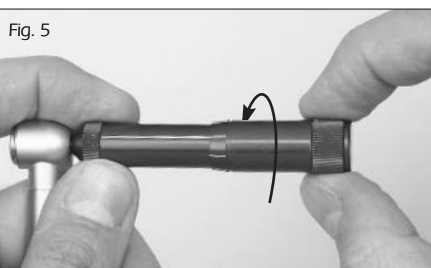
Fig. 5

Fig. 6

ACCESSORIES AND SPARE PARTS / ACCESSORI E RICAMBI / ZUBEHÖR UND ERSATZTEILE / ACCESSOIRES ET RÉCHANGES / ACCESORIOS Y RECAMBIOS / ACESSÓRIOS E PEÇAS SOBRESSELENTES / ΕΞΑΡΤΗΜΑΤΑ ΚΑΙ ΑΝΤΑΛΛΑΚΤΙΚΑ

REF 110.95	LubriONCE® Grease lubricator / Ingrassatore / Schmiernippel / Graisseur / Engrasador / Engraxador / Γρασαδόρος (2 pcs.)
REF 160.90	Rotor / Rotore / Rotor / Rotor / Rotor / Rotor / Ρότορας
REF 120.00	SO2000 Service Oil / Lubrificante / Schmiernippel / Lubrifiant / Lubricante / Lubrificante / Λάδι Λιπανής
REF 120.02	Nozzle for rapid connection / Beccuccio per attacco rapido / Kanüle für Schnellkupplung / Buse pour connexion rapide / Boquilla para conexión rápida / Bocal para conexão rápida / Ταχυσύνδεσμος λιπανής
REF 129.40	Air pressure gauge for 4-hole connection / Manometro per attacco 4 vie / Manometer für 4-Wege-Anschluss / Manomètre pour raccordement 4 voies / Manómetro para enlace 4 orificios / Manómetro de 4 vías / Μανόμετρο για κορδόνι με 4 σπές
REF 316.60	GYROFLEX®L rapid coupling for δ-pin connection / Raccordo rapido per connessione δ vie / Schnellkupplung für elektrischen 4-Wege-Anschluss / Raccord rapid pour raccordement 4 voies / Enlace rápido para conexión δ orificios / Acoplamento rápido para conexão δ vías / Ταχυσύνδεσμος για χειρολαβές με φως
REF 316.62	GYROFLEX®LED rapid coupling for δ-pin connection / Raccordo rapido per connessione δ vie / Schnellkupplung für elektrischen 4-Wege-Anschluss / Raccord rapid pour raccordement 4 voies / Enlace rápido para conexión δ orificios / Acoplamento rápido para conexão δ vías / Ταχυσύνδεσμος για χειρολαβές με φως

ENGLISH

DESCRIPTION

High-torque ceramic ball-bearing turbine for professional dental use, compliant to ISO 7785-1 standard.
 Weight without coupling: 47 g.
 Maximum speed of rotation: 340000 rpm.
 Output illumination: 25000 lux.
 Maximum noise level: 58 dBA (@2.3 bar).
 Instrument classification: Type 2 according to ISO 7785-1.

This medical device meets the requirements of the European Directive 93/42 EEC (Class IIa).

CONTENTS

THERA® L 40SE turbine, a LubriONCE® grease lubricator, a wire cleaner.

CONNECTION

The THERA® L 40SE turbine must be used together with the GYROFLEX®L (REF 316.60) or GYROFLEX®LED (REF 316.62) rapid coupling (Fig. 1). These version can also be used together with the Multiflex® rapid couplings.

AIR PRESSURE

Recommended air pressure: 2.1 ÷ 2.3 bar (30.5 ÷ 33.5 psi).
 Air consumption: 36 NI/min (@2.3 bar).
 Check air pressure by means of a gauge (REF 129.40) connected between the turbine and the supply hose (Fig. 2).
 Supplied air must be perfectly dry and well-filtered, kept at constant pressure.

SPRAY

Three-jet type.
 Spray air pressure: 1.0 ÷ 4.0 bar (14.5 ÷ 58.0 psi).
 Water pressure: 0.8 ÷ 2.5 bar (11.6 ÷ 36.3 psi). Water must be well-filtered.
 The use of unfiltered hard water will lead to early blockage of the tubings and spray diffusers.
 To clean the spray ducts, use the wire cleaner supplied, as shown on Fig. 4.

LIGHTING

The THERA® L 40SE turbine is equipped with a highly-efficient light conductor. The light source is located in the GYROFLEX®L or GYROFLEX®LED coupling.

The turbine must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 standard.

BUR TYPE

Shank diameter: Ø1.60 mm, Type 3 according to ISO 1797.
 Maximum length: 25 mm, code 4 or 5 or 6 (6th digit) according to ISO 6360-1.
 Minimum fitting length of shank: 11 mm.

BUR CHANGING

The turbines are equipped with a sophisticated push-button chuck: press the button with the thumb and subsequently insert or extract the bur (Fig. 3).

When fitting on a bur make sure it is fully inserted.
 Before inserting a bur, make sure the bur shank is clean.
 The turbine must not be started without a bur inserted into the chuck.
The bur gets automatically gripped as soon as the turbine is running.

To avoid overheating of the push-button, which could cause burns, the button should not be pressed while the turbine is running. Internal tissues (tongue, cheeks, etc..) must then be protected by a retractor or a dental mirror.

LUBRICATION BY GREASE

The turbine must be lubricated with the supplied LubriONCE® grease lubricator daily.
 At the very beginning, remove the cap and turn the rear housing clockwise until you can see grease coming out from the rod.

To lubricate turbine, carry out the following operations:
 1) Push the button and insert the terminal rod of the grease lubricator into the turbine in place of the bur.
 2) Holding turbine and grease lubricator, turn lubricator rear housing one mark clockwise (Fig. 5). During the operation, hold the lubricator perfectly longitudinal to the turbine head, so as to prevent the terminal rod of the lubricator from bending.
 3) Remove the grease lubricator, insert the bur and run the turbine for a few seconds before operating.

The injected quantity of grease will be sufficient for one day's work, even if the turbine is repeatedly sterilized in autoclave.

Lubrication is basic to life of the handpiece: repeat it **ONCE A DAY** so that the instrument you have purchased can meet your requirements for long time.

LUBRICATION BY OIL

As an option to grease, it is possible to lubricate instrument by means of the SO2000 service oil (REF 120.00) carrying out the following operations:
 1) Disconnect turbine from the rapid coupling;
 2) Insert and screw the special nozzle (REF 120.02) onto the lubricant can;
 3) Holding turbine by means of a cloth, fully insert the nozzle into the turbine (Fig. 6). Lubricate by pushing once only for one second.

In order for the instrument you have purchased to be helpful in your profession for a very long time, carefully repeat the described operations **before every sterilization cycle and, anyway, at least TWICE A DAY.**

STERILIZATION

The turbine can be sterilized in autoclave at 135°C / 2 bar (275°F / 29 psi). Before sterilization, remove residues and carefully clean the instrument. The LubriONCE® grease lubricator and GYROFLEX® rapid couplings cannot be sterilized.
 The device is supplied "not sterile".

CLEANING AND DISINFECTION

Clean with a cloth soaked in a disinfectant containing:
 - glutaraldehyde up to 3%
 - O-phenylphenol
 - alcohols based on ethanol
 Products not recommended: disinfectants containing benzalkonium chloride or acetone.
 The instrument cannot be immersed in any kind of liquid.

SERVICE

In case of irregular noise, head vibrations, bur swinging or any other malfunction of the instrument, and for any overhaul and repair work, please contact your usual supplier, an authorized service centre, or directly to TKD.

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