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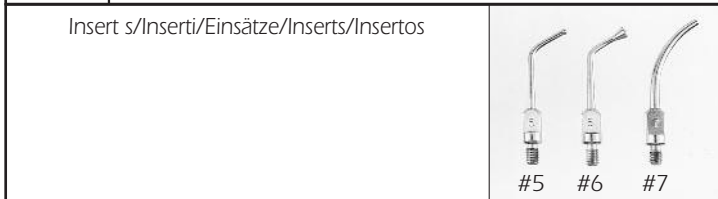
**DESCRIPTION** Pneumatic scaler. Tartar is removed by means of mechanical vibrations. The scaler meets ISO 15606 Standard. Weight without insert: 65 g. Frequency of vibrations: 6000 ÷ 6700 Hz. Maximum noise level: 78 dBA (@41 psi)

**CONTENTS** AIRSON scaler, Insert #1, Insert #2, Insert #4, a insert fitting key, a guarantee.

**CONNECTION** The AIRSON air scaler can be connected to any unit supplied with a standard turbine hose. The AIRSON scaler with quick connection (Fig. 2) must be used together with the MULTIWAY quick couplings (REF 315.10 or REF 315.20), whereas the other models with 3-hole or 4-hole connection (ISO 9168 Standard) can be directly connected to the unit hoses.

**AIR PRESSURE** Recommended air pressure is 29 ÷ 41 psi (2.0 ÷ 2.8 bar). Maximum air consumption is 18 NI/min (@41 psi). Check air pressure by means of a gauge (REF 335.10 or REF 335.20) connected between the handpiece and the supply hose. Supplied air must be perfectly dry and well-filtered, kept at constant pressure. Impurity or residues in the compressed air may cause the scaler to malfunction.

**COOLING** Operating water pressure for cooling spray must be 0.5 ÷ 2.5 bar. Water flow rate is higher than 50 ml/min (@2.0 bar). Water must be well-filtered.

**FITTING OF INSERTS** Select the right type of insert according to the operation to be performed and screw it manually onto the handpiece. The insert must then be moderately tightened using the supplied key (Fig. 1). Inserts must always be removed after use. Do not fit inserts while the scaler is running.

**POWER ADJUSTMENT** Power may be increased or decreased by rotating the ring nut located at the bottom of the handpiece (Fig. 3).

**MAINTENANCE OF INSERTS** The shape and weight of each insert are the result of a precise design optimisation so as to obtain maximum performance. Therefore any alteration, breaking, twisting or wearing of the insert can cause abnormal operation: in such case it is strongly recommended that the insert be replaced.

**UTILISATION OF SUPPLIED INSERTS (Fig. 4)**

INSERT #1	UNIVERSAL Treatment of the interdental joint. Treatment of the neck and subgingival region.
INSERT #2	EXTRA FINE Treatment of periodontal pouches. Interproximal treatment.
INSERT #4	SPATULA Treatment of major stones.

**OPTIONAL INSERTS** These inserts must be used for dry procedures (without spray cooling).

INSERT #5 & #6	Amalgam condensation.
INSERT #7	Crowns and pivots loosening.

**MAINTENANCE** In order for the instrument you have purchased to be helpful in your profession for a very long time, carefully repeat the following operation **daily**. In case of **long inactivity** however, it is recommended that this operation be always carried out **before resting the instrument**.

Detach the handpiece from the supply hose and lubricate using the special vaporiser (REF 220.95), as shown in Fig. 5. The SO2000 service oil (or any other lubricating oil for dental handpieces) must be inserted into instrument air input. Press once only. In case of scaler with quick connection, insert the nozzle reducer and lubricate as shown in Fig. 6.

Immediately after, screw the handpiece back to the hose (or reconnect it to the quick coupling) and operate the scaler for a few seconds, until the characteristic hissing sound can be heard.

**STERILIZATION** The handpiece, the key and inserts can be sterilized in autoclave at 135°C/2 bar (275°F/29 psi). The handpiece must always be sterilized without the insert on.

During sterilization, metal parts of different kinds should not come into contact with each other. To avoid this possibility, wrap the objects in a cloth or place them in a sterilisable bag.

The MULTIWAY quick couplings cannot be sterilized.

**CLEANING AND DISINFECTION** Clean with a cloth soaked in a disinfectant containing:

- glutaraldehyde up to 3%
- O-phenylphenol
- alcohols based on ethanol

Products not recommended: disinfectants containing benzalkonium chloride. The instrument cannot be immersed in any kind of liquid: inserts can be cleaned in a ultrasonic tank or with a germicidal cleanser. The screw thread can be properly cleaned with a cotton bud soaked in alcohol.

**SERVICE** In case of malfunction and for any overhaul and repair work, please contact your usual supplier or directly to TKD.

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