



GYROFLEX® & GYROFLEX® LED

REF 316.40
REF 316.62

Rapid couplings
Raccordi rapidi
Schnellkupplungen
Raccords rapides
Enlaces rápidos
Ασφάλματος rápidos
Ταχυσύνδεσμος

Operating instructions
Istruzioni per l'uso
Bedienungsanleitung
Mode d'emploi
Modo de empleo
Instruções de uso
Οδηγίες χρήσεως

CE

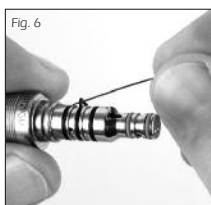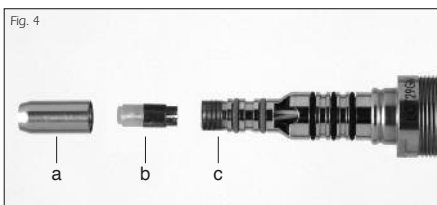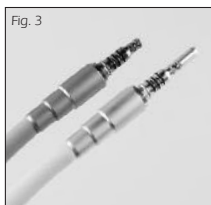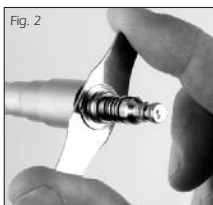ACCESSORIES AND SPARE PARTS / ACCESSORI E RICAMBI / ZUBEHÖR
UND ERSATZTEILE / ACCESSOIRES ET RÉCHANGES / ACCESORIOS
Y RECAMBIOS / ACCESÓRIOS E PEÇAS SOBRESSELENTES /
ΕΞΑΡΤΗΜΑΤΑ ΚΑΙ ΑΝΤΑΓΜΑΤΙΚΑ

REF 001.29.01 Green O-rings / Guarnizioni verdi / Grüne
O-Ringe / Joints toriques vertes / Guarniciones
verdes / O-rings verdes / Πράσινοι ελαστικοί
δακτύλιοι (10 pcs.)

REF 001.29.02 Black O-rings / Guarnizioni nere / Schwarze
O-Ringe / Joints toriques noires / Guarniciones
negras / O-rings pretos / Μαύροι ελαστικοί
δακτύλιοι (10 pcs.)

ENGLISH

DESCRIPTION

Rapid couplings for use with all quick-release TKD turbines or with any other dental instrument with Multiflex® connection (Fig. 1). Couplings are equipped with a non-return valve for water.

Maximum operating pressure of air and water: 6 bar (87 psi).

The GYROFLEX®LED version, with integrated LED light source (3.5 ± 0.2 Vdc; 60 mA maximum), provides for turbine illumination.

This medical device meets the requirements of the Directive 93/42 EEC.

CONTENTS

GYROFLEX® or GYROFLEX®LED rapid coupling, tool for coupling installation, set of spare O-rings.

CONNECTION TO THE SUPPLY HOSE

Screw the GYROFLEX® or GYROFLEX®LED coupling onto the flexible supply hose. The connector meets ISO 9168 Standard.

For a perfect locking, use and insert the supplied tool on the corresponding grooves (Fig. 2).

The coupling, then, need not be removed from the hose (Fig. 3).

Couplings must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 Standard.

Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

LED POSITIONING

The LED must be supplied with the right polarity. If the LED does not turn on: unscrew the protection cap (a), pull out the LED (b), rotate it by 180° so as to reverse polarity on its contacts and insert it again on its housing (c) (Fig. 4). Screw cap again.

O.R. MAINTENANCE

In order to increase coupling seal and smoothness, we recommend that you regularly rub the O-rings with a little quantity of non-toxic grease. The grease can also be taken from the turbine LubriONCE® grease lubricator (Fig. 5).

O.R. REPLACEMENT

Should water or air leakage occur, replacement of O-rings on the coupling is recommended.

With the aid of a pin, pull out the old O-rings and insert the new ones on the corresponding grooves (Fig. 6).

CLEANING AND DISINFECTION

Couplings can be externally disinfected with a non-chlorinated germicidal cleanser. Additionally, the cleanser must not contain any glutaraldehyde, acid or solvent.

Rapid couplings cannot be sterilized.

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