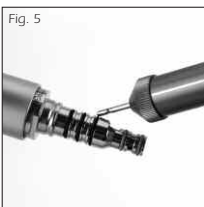
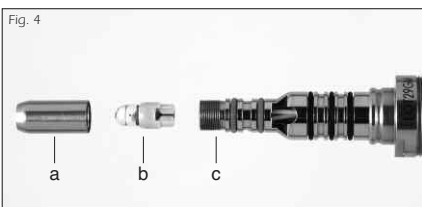
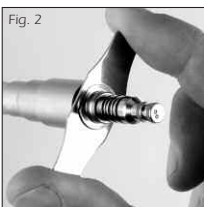
GYROFLEX® & GYROFLEX®L

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ENGLISH

DESCRIPTION

Rapid couplings for use with all quick-release TKD handpieces or with any other dental instrument with Multiflex® connection (Fig. 1). Couplings are equipped with a non-return valve for water.

Maximum operating pressure of air and water: 6 bar (87 psi).

The GYROFLEX®L version, with integrated light source (3.3 ± 0.2 V; 700 mA; Average life: 80h @3.3 V), provides for turbine illumination.

This medical device meets the requirements of the Directive 93/42 EEC.

CONTENTS

GYROFLEX® or GYROFLEX®L coupling, tool for coupling connection, set of spare O-rings, a spare bulb (only with the GYROFLEX®L coupling).

CONNECTION TO THE FLEXIBLE HOSE

Screw the GYROFLEX® or GYROFLEX®L coupling onto the flexible supply hose. The connector meets ISO 9168 Standard.

For a perfect locking, use and insert the supplied tool on the corresponding grooves (Fig. 2).

The coupling, then, need not be removed from the hose (Fig. 3).

Couplings must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 Standard.

Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

BULB REPLACEMENT

Unscrew the bulb protection cap (a), pull out the old bulb (b) and properly insert the new one on its housing (c) (Fig. 4). Screw cap again.

O.R. MAINTENANCE

In order to increase coupling seal and smoothness, we recommend that you regularly rub the O-rings with a little quantity of non-toxic grease. The grease can also be taken from the turbine grease lubricator (Fig. 5).

O.R. REPLACEMENT

Should water or air leakage occur, replacement of O-rings on the coupling is recommended.

With the aid of a pin, pull out the old O-rings and insert the new ones on the corresponding grooves (Fig. 6).

CLEANING AND DISINFECTION

Couplings can be externally disinfected with a non-chlorinated germicidal cleanser. Additionally, the cleanser must not contain any gluteraldehydes, acid or solvent. Rapid couplings cannot be sterilized.

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