
TeKne Dental

# Hose for electric micromotors Tubo per micromotori elettrici Schlauch für Elektro-Mikromotoren Tuyau pour moteurs électriques Manguera para motores eléctricos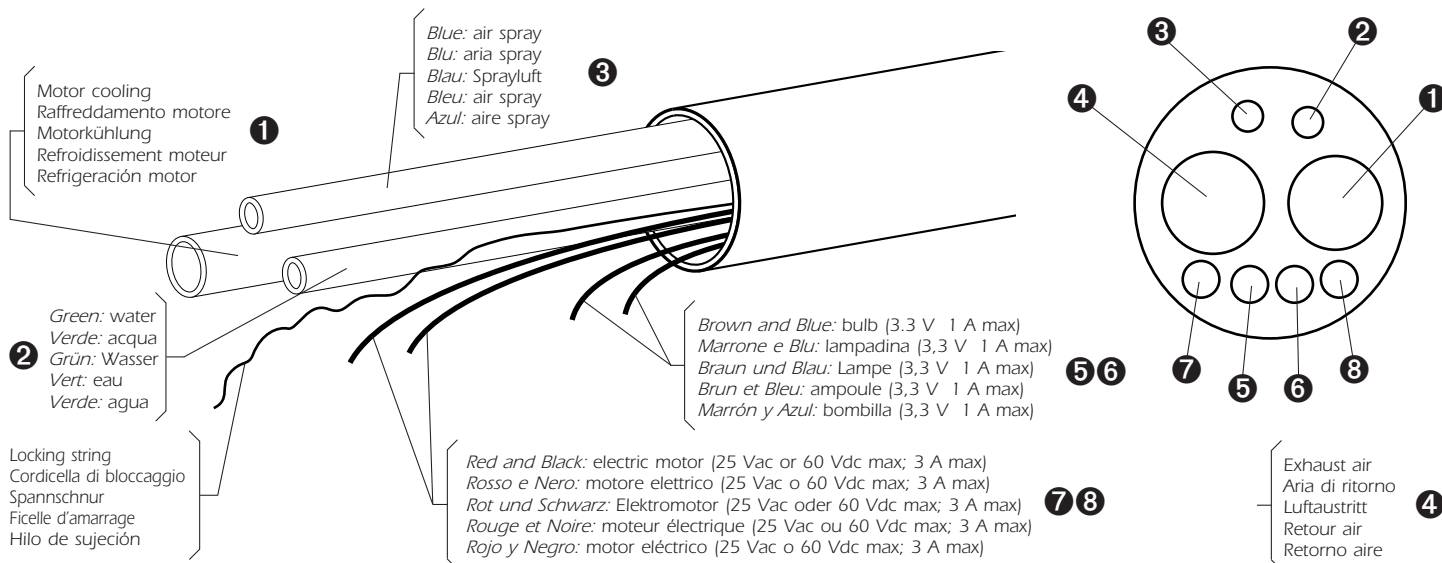

## Connection

- The hose is suitable for use with electric micromotors meeting the ISO 11498 Standard and having Ø15 x 0.5 mm. connector thread.
- Standard length of the hose: 1.6 m.
- Hose diameter: 10,5 mm.
- Maximum operating pressure of spray ducts and cooling air: 6 bar (87 psi).
- Hose must be connected to the power supply unit meeting the requirements of IEC 601-1 Standard. Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

## Assembly

- Keep the initial alignment of the wires and tubings.
- Fix and lightly pull the locking string.
- To connect electric wires correctly, refer to the connection diagram for the electronic control board of the micromotor.

## Maintenance

- Clean with a cloth soaked in a disinfectant containing:
  - glutaraldehydes up to 3%
  - alcohols based on ethanol
- To keep the surface of the hose in good conditions, it is advisable to periodically wipe it with talcum powder.
- The hose is not sterilisable.

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