
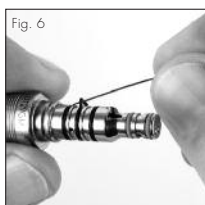
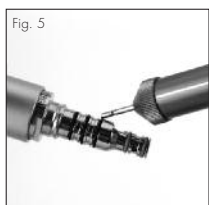
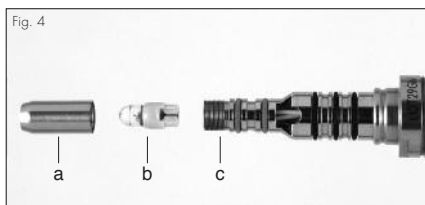
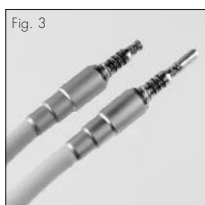
GYROFLEX® & GYROFLEX®L

REF 316.30
REF 316.40
REF 316.60

Rapid couplings
Raccordi rapidi
Schnellkupplungen
Raccords rapides
Enlaces rápidos
Acoplamentos rápidos
Ταχυσύνδεσμος

Operating instructions
Istruzioni per l'uso
Bedienungsanleitung
Mode d'emploi
Modo de empleo
Instruções de uso
Οδηγίες χρήσεως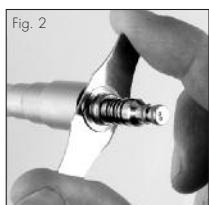

ACCESSORIES AND SPARE PARTS / ACCESSORI E RICAMBI / ZUBEHÖR
UND ERSATZTEILE / ACCESSOIRES ET RÉCHANGES / ACCESORIOS
Y RECAMBIOS / ACCESSÓRIOS E PEÇAS SOBRESSELENTEIS /
ΕΞΑΡΤΗΜΑΤΑ ΚΑΙ ΑΝΤΑΛΛΑΚΤΙΚΑ

REF 028.00.00	Xenon bulb / Lampadina allo Xenon / Lampe Xenon / Ampoule Xenon / Bombilla Xenon / Lámpada Xenon / Λαμπτήρα τύπου Xenon (2 pcs.)
REF 001.29.01	Green O-rings / Guarnizioni verdi / Grüne O-Ringe / Joints toriques vertes / Guarniciones verdes / O-rings verdes / Πράσινοι ελαστικοί δακτύλιοι (10 pcs.)
REF 001.29.02	Black O-rings / Guarnizioni nere / Schwarze O-Ringe / Joints toriques noirs / Guarniciones negras / O-rings pretos / Μαύροι ελαστικοί δακτύλιοι (10 pcs.)

ENGLISH

DESCRIPTION

Rapid couplings for use with all quick-release TKD handpieces or with any other dental instrument with Multiflex® connection (Fig. 1). Couplings are equipped with a non-return valve for water.

Maximum operating pressure of air and water: 6 bar (87 psi).

The GYROFLEX®L version is provided with an integrated Xenon bulb (3.5 ± 0.2 V; 740 mA max) for illumination.

This medical device meets the requirements of the European Directives 93/42/EEC (Class I) and 2007/47/EC.

CONTENTS

GYROFLEX® or GYROFLEX®L coupling, tool for coupling connection, set of spare O-rings, a spare bulb (only with the GYROFLEX®L coupling).

CONNECTION TO THE FLEXIBLE HOSE

Screw the GYROFLEX® or GYROFLEX®L coupling onto the flexible supply hose. The connector meets ISO 9168 Standard.

For a perfect locking, use and insert the supplied tool on the corresponding grooves (Fig. 2).

The coupling, then, need not be removed from the hose (Fig. 3).

Couplings must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 Standard.

Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

BULB REPLACEMENT

Unscrew the bulb protection cap (a), pull out the old bulb (b) and properly insert the new one on its housing (c) (Fig. 4). Screw cap again.

O.R. MAINTENANCE

In order to increase coupling seal and smoothness, we recommend that you regularly rub the O-rings with a little quantity of non-toxic grease. The grease can also be taken from the turbine grease lubricator (Fig. 5).

O.R. REPLACEMENT

Should water or air leakage occur, replacement of O-rings on the coupling is recommended.

With the aid of a pin, pull out the old O-rings and insert the new ones on the corresponding grooves (Fig. 6).

CLEANING AND DISINFECTION

Couplings can be externally disinfected with a *non-chlorinated* germicidal cleanser. Additionally, the cleanser must not contain any *glutaraldehydes, acid or solvent*. Rapid couplings cannot be sterilized.

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