
# Hose for electric micromotors

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# Manguera para motores eléctricos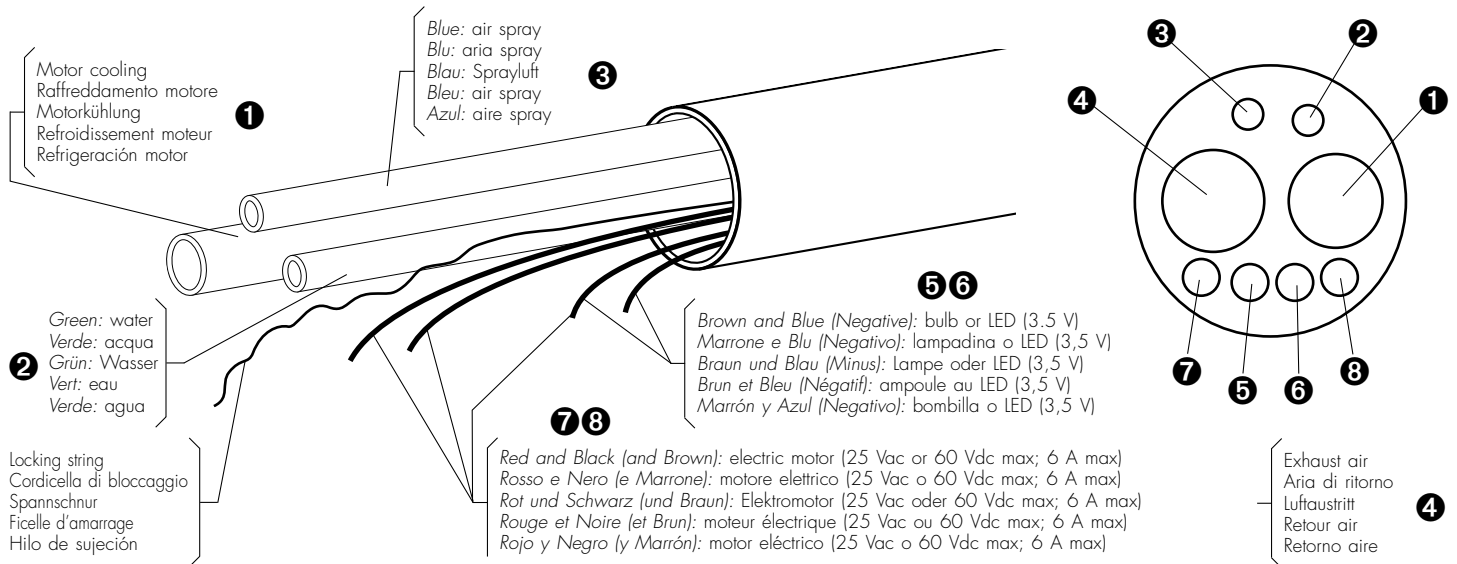

## Description

- The hose is suitable for use with the Bien-Air MC3/LED and TKD DEFINITIVE®/LED electric micromotors.
- Maximum operating pressure of air and water: 6 bar (87 psi).
- Standard length of the hose: 1,6 m.
- Hose diameter: 10,5 mm.
- This medical device meets the requirements of the European Directives 93/42/EEC (Class I) and 2007/47/EC.

## Assembly

- Keep the initial alignment of the wires and tubings.
- Fix and lightly pull the locking string.
- Hose must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 Standard. Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

## Maintenance

- Clean with a cloth soaked in a disinfectant containing:
  - glutaraldehyde up to 2%
  - alcohols based on ethanol or methanol
 Finally wipe with a humid cloth.
- Disinfectants containing acetone, chlorine, bleach or butoxyethanol are not recommended.
- To keep the surface of the hose in good conditions, it is advisable to periodically wipe it with talcum powder.
- The hose is not sterilisable.

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