

Improved air scaler for easy tartar removal.

The scaler can be used for all prophylaxis procedures using the supplied TKD's inserts or with the aid of the special KaVo/Komet inserts.

The scaler can be used on any unit, in place of the usual high-speed turbine, without requiring any additional adjustment.

Vibration power level can be adjusted by means of a larger rotating nut, making the prophylaxis more flexible and accurate.

Scaler is available with standard 3-hole or 4-hole connection.



Supplied inserts are: Universal (#1), Extra-fine (#2) and Spatula (#4).

TECHNICAL DATA

| | TITANU5®E | TITANUS®S |
|----------------------|---|-------------------|
| Operating frequency: | 27 ÷ 32 KHz | 28 ÷ 32 KHz |
| Sterilization: | up to 134°C/2 bar | up to 134°C/2 bar |
| Weight: | 45 g | 45 g |
| Standard conformity: | ISO 22374, CEI EN 60601-1, CEI EN 60601-1-2, IEC 80601-2-60 European Directives 93/42/CEE (Class IIa) e 2007/47/CE | |
| Guarantee: | 2 years | 2 vears |

| | <i>USC60</i> | |
|----------------------|--|--|
| Input voltage: | 24 Vac ± 10% | |
| Maximum consumption: | 1 A, 24 VA | |
| Operating frequency: | 27 ÷ 32 KHz | |
| Output power: | 0.2 ÷ 9 W | |
| Sizes: | 60 x 54 x 30 mm | |
| Standard conformity: | CELEN 60601-1, CELEN 60601-1-2, IEC 80601-2-60 | |
| Guarantee: | 2 years | |

| | <i>PIRSON</i> ® | |
|--------------------------|---|--|
| Operating frequency: | 6 ÷ 7 KHz | |
| Drive air pressure: | 2.4 ÷ 3.0 bar | |
| Maximum air consumption: | 22 NI/min | |
| Sterilization: | up to 134°C/2 bar | |
| Weight: | 64 g | |
| Standard conformity: | ISO 15606, ISO 9168, European Directives 93/42/CEE and 2007/47/CE | |
| Guarantee: | 1 year | |

Piezon® is a registered trademark of Electro Medical Systems SA, Switzerland Newtron® is a registered trademark of Acteon SA, France.










TeKne Dental srl • Via del Pescinale, 77 • 50041 Calenzano (FI) • Italy

\$\psi\$ +39 055 8825741 • **\$\pi\$** +39 055 8825764

www.teknedental.com • info@teknedental.com

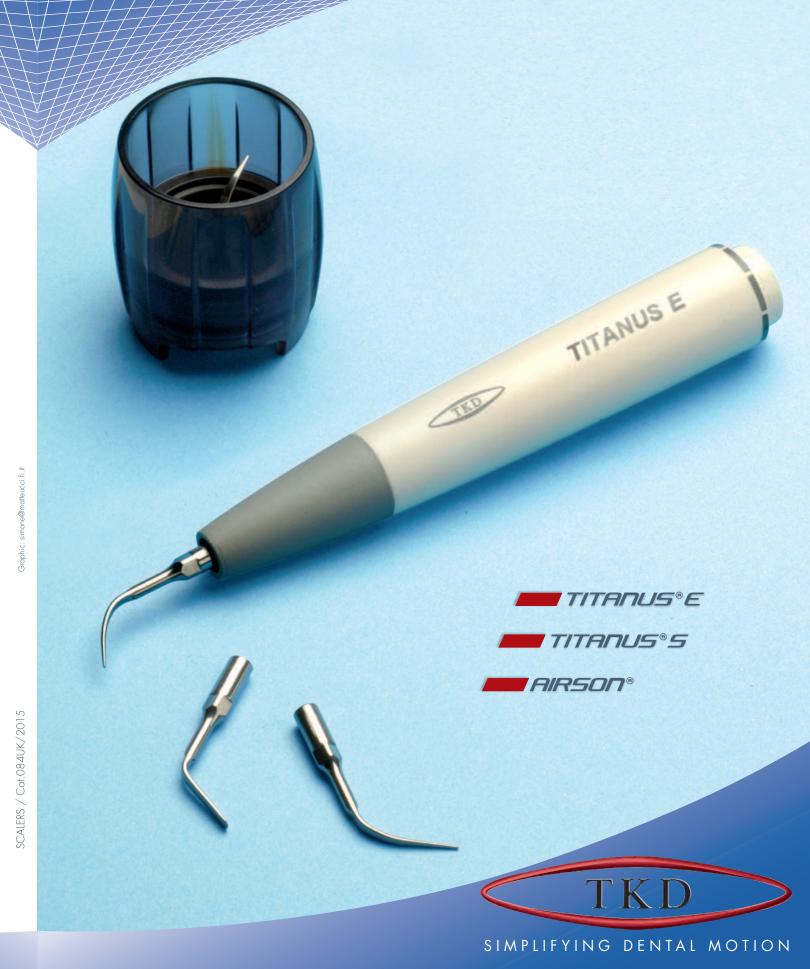

TITANU5®E

ultrasonic scaler

The improved scaler now features higher reliability and greater power efficiency, allowing a powerful cavitation effect at the tip of the inserts.

Due to the piezo-electric technology and to the perfectly aligned titanium transducer, which has been optimized for use with EMS®-type inserts, generated mechanical vibrations are absolutely linear and longitudinal to the handpiece axis. The scaler is available in two different versions: with TKD special connection or with Piezon®-compatible connection.

Supplied inserts are made in biocompatible high-quality surgical steel. The surface of the inserts is extremely fine thanks to the special polishing process which guarantees low wear and extended lifetime. Inner thread is extremely precise so as to achieve perfect coupling and safeguard the titanium transducer.



A version, with Piezon®-compatible connection, is also available.



The DYNO dynamometric wrench allows fast and precise locking of the inserts.



The DYNO dynamometric wrench allows safe handling and sterilisation of the inserts.







INSERT # DX

EMS®-compatible

Use: removal of crowns, bridges

and pivots.

INSERT # EX

Special 120°

diameter.

EMS®-compatible

ENDO chuck for use

with ultrasonic files

with 0.8 mm shank

INSERT # PLX3

EMS®-compatible

Use: for cleaning

and disinfecting

periodontal

pockets.



EMS®-compatible Use: removal of supragingival tartar, coatings and stains on all tooth surfaces.

INSERT # PX

EMS®-compatible

subgingival tartar

deposits on all tooth

surfaces, including

interproximal and

ulcus areas.

INSERT # PLX1

EMS®-compatible

Use: left-curved

debridement on

hard-to-access

INSERT # PLX4

Use: left-curved

insert with small

ball on tip for

debridement on

urcations and

periodontal

EMS®-compatible

insert for

periodontal

Use: removal of

INSERT # BX



INSERT # CX EMS®-compatible Use: removal of heavy supragingival deposits of tartar.







INSERT # PLX2 EMS®-compatible Use: right-curved insert for periodontal debridement on hard-to-access



INSERT # PLX5 EMS®-compatible Use: right-curved insert with small ball on tip for periodontal debridement on furcations and concavities.



INSERT # BX-S Satelec®-compatible Use: removal of supragingival tartar, coatings and stains on all tooth surfaces.

TKD TITANUS S



INSERT # PX-S Satelec®-compatible Use: removal of subgingival tartar deposits on all tooth surfaces, including interproximal and sulcus areas.



ultrasonic scaler control board

TITANU5°5

Newtron®-compatible connection.

ultrasonic scaler


This new model has been optimized for use with Satelec®-type inserts.

The scaler is available in two different versions: with TKD special connection or with

A new version, with Newtron®-compatible connection, is also available.

The improved USC60 ultrasounds generator delivers gentle sinusoidal output signal which, combined with the linear oscillating action and uniform surface smoothness of the inserts, allows to achieve high precision and patient comfort.



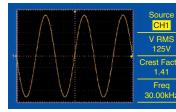

The generator allows greater efficiency and optimum power control for all applications: Endodontics, Periodontics, Scaling, Restorative.

There is no need for external switches, a single potentiometer or control voltage enables to cover all the clinical range.







