*TeKne Dental*

**APPLICATION NOTES**

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**January 2005**

Gennaio 2005

**PLC2 ELECTRONIC BOARD**

SCHEDA ELETTRONICA PLC2

*TeKne Dental* s.r.l.

Via del Pescinale 77 - 50041 Calenzano (FI) - ITALY  
info@teknedental.com      www.teknedental.com

**DESCRIPTION**

The PLC2 electronic board (REF 382.00) provides a universal power supply for two bulbs of all types of fibre-optic instruments. Each bulb can be turned on by connecting the instrument's air drive to the pneumatic switch integrated in the board. Alternatively, external switches can be used. Activation of LAMP 1 takes priority over the other.

Input voltage has a wide range and can be either alternative or direct, whereas output voltage (lamp power supply) is direct and can be adjusted by means of a trimmer.

By removing one jumper bulb turn-off time can be delayed up to 10 seconds from the corresponding switch aperture.

**ASSEMBLY**

The board can be fixed to the desired location by means of the supplied strap tape.

**CONNECTION**

To connect board properly, please follows wiring diagrams shown on Fig. 2 or 3:

- Fig. 2 shows the diagram for power supply of two bulbs through the integrated pneumatic switches;
- Fig. 3 shows the diagram for power supply of two bulbs through external electric switches.

To protect the board, it is necessary to connect an external fuse (F 250 mA) as shown. Board must be connected to a power trasformer having a minimum insulation strenght of 500 V between secondary winding and earth and a minimum insulation strenght of 4000 V between secondary winding and mains.

The manufacturer will not be held responsible in case of use on a electric connection not meeting the requirements of IEC 601-1 Standard.

**SPECIFICATIONS**

- Input voltage: 14 ÷ 26 Vac or 19 ÷ 36 Vdc.
- Output voltage: 2.6 ÷ 3.8 Vdc.
- Maximum output current: 1 A
- Maximum consumption: 3.1 W (Bulb) + 1.1 W (Board)
- Lamp turn-off delay time: 5 s or 10 s selectable by a jumper.
- External protection fuse: F 250 mA
- Circuit activation by means of integrated pneumatic switches.
  - Minimum activation pressure: 1.4 ± 0.4 bar (20.3 ± 5.8 psi)
  - Maximum operating pressure: 4 bar (58 psi)
  - Absolute maximum rating: 6 bar (87 psi)
- Board sizes: 72 x 50 x 30 mm
- Working conditions: Temperature: 0 ÷ 40 °C (32 ÷ 104 °F); Humidity: 30 ÷ 90 %
- Storage conditions: Temperature: -20 ÷ 100 °C (-4 ÷ 212 °F); Humidity: 10 ÷ 90 %
- Guarantee: 2 years.

**STANDARDS REFERENCE**

The board does not produce electromagnetic interference and meets the requirements of IEC 601-1 and IEC 601-1-2 Standards.

**SERVICE**

In case of malfunction and for any overhaul and repair work, please contact an authorized technician or directly to TKD.

The manufacturer will not be held responsible in case of operations or repairs carried out by technicians not authorized by the manufacturer.

**ACCESSORIES AND SPARE PARTS**

REF 095.10	PLASTIC 3-WAY CONNECTOR (3 x 1/16")	Pack of 10 pieces
REF 095.11	PLASTIC 3-WAY CONNECTOR (2 x 1/8" + 1 x 1/16")	Pack of 10 pieces

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