REF 430.1n

INSERT #PLXn
INSERTO #PLXn

TeKne Dental s.r.l.
Via del Pescinale, 77 - 50041
Calenzano (FI) - Italy
info@teknedental.com
www.teknedental.com

TKD430.1n/Rev.01.2022

ENGLISH

Insert line for Periodontics

INSERT #PLX1 Use: left-curved model for periodontal debridement on hard-to-access approximal surfaces.

INSERT #PLX2 Use: right-curved model for periodontal debridement on hard-to-access approximal surfaces.

INSERT #PLX3 Use: model for the irrigation and disinfection of periodontal pockets with antimicrobial solution.

INSERT #PLX4 Use: left-curved model with ball-shaped tip for debridement in root furcations and concavities.

INSERT #PLX5 Use: right-curved model with ball-shaped tip for debridement in root furcations and concavities.

Power setting (min. ÷ max.): 1 ÷ 7
Irrigation flow rate: medium ÷ high
Never direct the tip of the insert towards the tooth surface.

The Insert is compatible with the TITANUS[®]E (REF 400.00.OX and REF 410.00.OX), TITANUS[®]ELED (REF 412.00.OX) and EMS piezo-electric Scalers.
To remove or fix the Insert onto the scaler, use the supplied wrench.
To remove or fix the Insert onto the scaler, use the special DYNO dynamometric wrench (REF 109.00).
For use, cleaning and sterilization of the Insert, please refer to the scaler's operating instructions.

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