

REF 430.08



INSERT #EX

INSERTO #EX



TeKne Dental s.r.l.

Via del Pescinale, 77
50041 Calenzano (FI) - ITALY

TKD430.08/Rev.09.2018

ENGLISH

Special 120° ENDO chuck for use with ultrasonic files with 0.8 mm shank diameter.

The Insert is compatible with the TITANUS®E (REF 400.00.OX and REF 410.00.OX) and EMS® piezo-electric Scalers.

To remove or fix the Insert onto the scaler, use the supplied wrench.

To remove or fix the files onto the ENDO chuck, use the special DYNO dynamometric wrench (REF 109.00).

For use, cleaning and sterilization of the Insert, please refer to the scaler's operating instructions.

ITALIANO

Speciale pinza ENDO a 120° per l'uso con le lime ultrasoniche con diametro del gambo di 0,8 mm.

L'Inserto è compatibile con gli ablatori piezo-elettrici TITANUS®E (REF 400.00.OX e REF 410.00.OX) e EMS®.

Per rimuovere o bloccare l'Inserto sull'ablatore, usare la chiavetta in dotazione.

Per rimuovere o bloccare le lime sulla pinza ENDO, usare la speciale chiavetta dinamometrica DYNO (REF 109.00).

Per l'uso, la pulizia e la sterilizzazione dell'Inserto, riferirsi alle istruzioni operative dell'ablatore.