SONOSURGERY® AIR SURGICAL UNIT

OPERATION AND MAINTENANCE MANUAL

ENGLISH

INTENDED USE

PNEUMATIC SURGICAL Unit for control and automatic sterile irrigation of the SONOSURGERY® AIR POWER vibrating handpiece.

The SONOSURGERY® Unit allows optimal control of pneumatic vibrating handpieces and their respective inserts for surgery. It can be used in oral surgery for the collection of particulate bone and blocks, excision of cysts, preparation of implant sites, extraction of included teeth, germectomies, maxillary sinus lifts with crestal or lateral access, corticotomies, osteotomies, osteoplasty, etc...).

The SONOSURGERY® Unit can only be used by qualified surgeons.

This medical device meets the requirements of European Directives 93/42/EEC and 2007/47/EC.

TABLE OF CONTENTS

- INTRODUCTION	3
- OPERATION	3
- CONTENTS OF THE SUPPLIED SUITCASE	3
- ACCESSORIES AND SPARE PARTS PURCHASED SEPARATELY	5
- SAFETY PRECAUTIONS	5
- UNIT INSTALLATION	6
- FRONT PANEL CONNECTIONS AND CONTROL FUNCTIONS	9
- REAR PANEL CONNECTIONS	10
- ACTIVATION OF HANDPIECE AND STERILE IRRIGATION	10
- CONNECTION, USE AND STERILIZATION OF HANDPIECE AND COUPLING	10
- UNIT CLEANING AND DISINFECTION	10
- GUARANTEE	11
- TECHNICAL ASSISTANCE	11
- REFUSE DISPOSAL AT THE END OF LIFE	11
- TECHNICAL SPECIFICATIONS	11

SONOSURGERY® devices manufactured by:

TeKne Dental s.r.l.

Via del Pescinale, 77
50041 Calenzano (FI) - Italy
info@teknedental.com
www.teknedental.com

Congratulations on the choice!

This pneumatic surgery device has been designed to provide many years of reliable service. Please read carefully and follow the instructions provided in this manual to get a long and safe service from your TKD instrument.

To illustrate the operation and maintenance of handpieces, couplings, inserts or other accessories for your Unit, refer to the specific manuals supplied separately.

INTRODUCTION

Your new AIR SURGICAL UNIT is an innovative pneumatic device (patent No. WO 2018/172807) available for the dental profession. Together with the powerful AIR POWER handpiece, the exclusive STERIL rapid coupling and the special surgical sonic inserts, it completes the SONOSURGERY® vibrating surgical system.

The Unit has been intentionally designed with pneumatic components only and therefore does not contain any electrical component. This ensures greater safety and the SONOSURGERY® system can be used safely on pacemaker wearers and can also be used in operating theaters even in the presence of flammable anesthetic gases.

The Unit combines the control of the powerful AIR POWER vibrating handpiece and the supply of its sterile irrigation with a simple and innovative automatic system.

The Unit can be used with all commercial pneumatic vibrating and rotating handpieces, provided they are equipped with a Multiflex®-compatible connection.

OPERATION

SONOSURGERY® AIR SURGICAL UNIT is a device for the control, operation and automatic supply of sterile irrigation, for the SONOSURGERY® AIR POWER handpiece, a pneumatic handpiece that produces orbitating movements at sonic frequencies of about 6000 Hz.

The pneumatically operated sonic system should not be confused with the piezoelectric ultrasonic system: the two vibrating instruments have a different power supply and a different movement.

Through its handpiece, the sonic vibrating system allows to generate complex orbitating oscillations of the inserts, creating three-dimensional movements of each point of their entire surface. The peculiar movement of the AIR POWER handpiece, allows the construction of inserts with infinite angles using all the space planes.

CONTENTS OF THE SUPPLIED SUITCASE

- Main Unit (Fig.1).
- Braided rubber hose for compressed air connection, equipped with quick male and female connectors at both ends (Camozzi 5000 Mini DN5 series) (Fig.2).
- Pneumatic pedal with air tubings (Fig.3).
- Steel arch with hook for supporting the pressure bag (Fig.4).
- Handpiece holder adjustable support for the arch (Fig.5).
- Inflatable pressure bag with integrated steel reinforcement (Fig.6).
- Silicone cord for handpiece supply, comprising connectors with ring nuts that can be screwed to both ends (Fig.7).
- SONOSURGERY® STERIL rapid coupling (Multiflex®-compatible) for sterile irrigation (Fig.8).
- A disposable set for sterile irrigation (Fig.9).
- Intro kit comprising 4 sonic inserts for surgery (Fig.10).

Fig. 1

Fig. 5

Fig. 2

Fig. 6

Fig. 3

Fig. 7

Fig. 4

Fig. 8

Fig. 9

Fig. 10

ACCESSORIES AND SPARE PARTS PURCHASED SEPARATELY

- SONOSURGERY® AIR POWER vibrating handpiece with dynamometric wrench
- Range of surgical sonic inserts (SBM0x, SSB0x, SSBFx, SSL0x, SBS0x, SIS0x, etc.)
- Disposable sterile irrigation set

SAFETY PRECAUTIONS

TKD assumes no responsibility for direct or indirect damages or damages resulting from improper use, in particular through failure to follow the instructions for use, or improper preparation and maintenance of this product.

WARNING: Use the Unit for its intended purpose only. Failure to follow the instructions for use can result in serious injury to the patient or user, or the product may be damaged, even irreparably. Before using this product, make sure you have read and understood the operating instructions.

WARNING: Current regulations restrict the sale of this device by or on the order of a dentist. The SONOSURGERY® Unit can in fact only be used by qualified surgeons.

WARNING: The Unit and its components are supplied non-sterile. Before first use and then before each patient, sterilize or disinfect the specified components as recommended in the "Sterilization" and "Disinfection" sections.

WARNING: The device can be used without risk even on people with pacemakers.

WARNING: Do not disassemble or modify the Unit, the handpiece, the rapid coupling or the pedal.

ATTENTION: The Unit can be used with all pneumatic vibrating and rotating handpieces on the market, such as: turbines, air motors, air scalars, etc. as long as they are compatible with the SONOSURGERY® STERIL connection (Multiflex®-compatible). The use of other dental accessories or assembled by third parties is the sole responsibility of the user.

ATTENTION: All repairs must be carried out by authorized TKD service personnel only.

ATTENTION: The irrigation system has been designed for use with sterile water. However, it is also possible to use a saline solution but in this case, to avoid encrustments, it is necessary to rinse the SONOSURGERY® STERIL device abundantly with water immediately after each use.

ATTENTION: Never use damaged or worn inserts as they could fracture or overheat hard and soft tissues.

UNIT INSTALLATION

Remove the Unit from the case and place it on a flat surface of a table or trolley.

Keep the original case for storage and transport.

With reference to the various figures, perform the following steps:

- 1) Fit the steel arch (Fig.4) by inserting the two vertical rods into the two side supports on the Unit (Fig.11), making sure the hook on the horizontal rod is facing forward. Attach the handpiece support (Fig.5) on the left or right side according to preference.
- 2) Insert a soft disposable bag (Fig. 12) of sterile irrigating liquid (saline solution or water) from the plastic eyelet side, pushing it all the way into the mesh pocket of the pressure bag (Fig. 13).
- 3) Make sure that the roller clamp is closed (Fig.14) and only then insert the bayonet punch into the bag (Fig.15), so as to avoid uncontrolled leakage of liquid from the irrigation set.
- 4) Hang the sterile liquid bag through the eyelet on the arch hook, so that the pressure bag gets supported by the bag it contains (Fig. 16).
- 5) Connect the female connector of the braided hose to the male inlet on the Unit (Fig.17).
- 6) Connect the male connector on the other end of the braided hose to a compressed air source (compressor, dental unit, etc.), fitted with a female quick coupling (Camozzi 5000 Mini DN5) to put the Unit under pressure (Fig.18).
- 7) Connect the male connector of the silicone cord to its female coupling on the front panel of the Unit (Fig. 19), screwing and tightening the metal nut. Then connect the female connector of the silicone cord to the SONOSURGERY® STERIL, always screwing and tightening its metal ring (Fig. 20).
- 8) Connect both pedal air tubings to the two pressure connectors on the front panel of the Unit (Fig. 21).
- 9) Turn the switch on the front panel to "ON" position (Fig. 22), which manually opens the tubing clamp valve, and position the silicone part of the irrigation set in its housing (Fig.23).
- 10) Insert the free end of the irrigation set into the appropriate inlet barb of the SONOSURGERY®STERIL connector (Fig.24).
- 11) Turn the switch back to "AUTO" position (Fig. 22), which closes the tubing clamp valve, in order to block the flow of the liquid. Only at this point can the roller clamp (Fig. 14) of the irrigation tubing be opened to allow the flow of the liquid to the handpiece, when the pedal is being pressed.
- 12) Connect the tubing of the pressure bag device to its quick connector on the rear panel (Fig. 25), so that the air coming out of the Unit can inflate it up to the pressure set by the front panel regulator (max. 0.6 bar).

Fig. 11

Fig. 14

Fig. 12

Fig. 15

Fig. 13

Fig. 16

Fig. 17

Fig. 21

Fig. 18

Fig. 22

Fig. 19

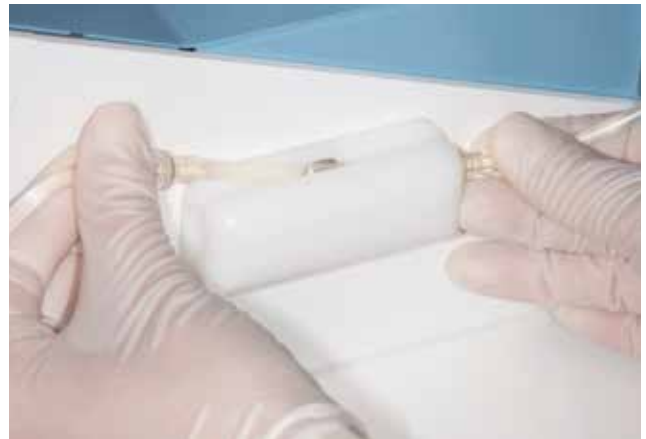
Fig. 23

Fig. 20

Fig. 24

Fig. 25

Note: The quick female connector, Camozzi 5000 Mini DN5 series (Fig 26), is not supplied by TKD and must be prepared on the unit, compressor or other by a qualified technician.

Fig. 26

FRONT PANEL CONNECTIONS AND CONTROL FUNCTIONS (Fig.1a)

a) Gauge for measuring air pressure supplied to the pressure bag.

b) Switch for setting the tubing clamp valve. When in "AUTO" position, the clamp valve opens automatically when the pedal is being pressed and closes when it is released. When in "ON" position, the clamp valve is forced open.

c) Adjustment knob for setting air pressure supplied to the pressure bag: the sterile liquid flow is adjusted by means of the knob, setting a pressure value at the bag from 0 to a maximum of 0.6 bar; a limiting valve prevents this limit from being exceeded. At 0.6 bar the maximum flow of the irrigating liquid is about 50 mL / min.

d) Pressure gauge to measure the air pressure of the handpiece.

e) Adjustment knob for setting air pressure supplied to the handpiece: the maximum pressure can be set using the knob with values from 0 to a maximum of 4.2 bar (*) according to the needs of the operator. The maximum set pressure can be reached gradually by progressively adjusting the thrust of the foot on the control pedal.

f) Air connection (exit / exhaust) for the handpiece silicone cord.

g) Push-in couplings for the pedal air tubings (inlet / outlet).

(*) The maximum value of the operating air pressure must be established with the rotating ring of the AIR POWER handpiece fixed at level 3 (with levels 2 or 1 the value of the measured pressure can slightly increase).

Note: The power of the handpiece must be set using its rotating ring: level 1 for minimum power, level 2 for medium power, level 3 for maximum power (see corresponding user manual).

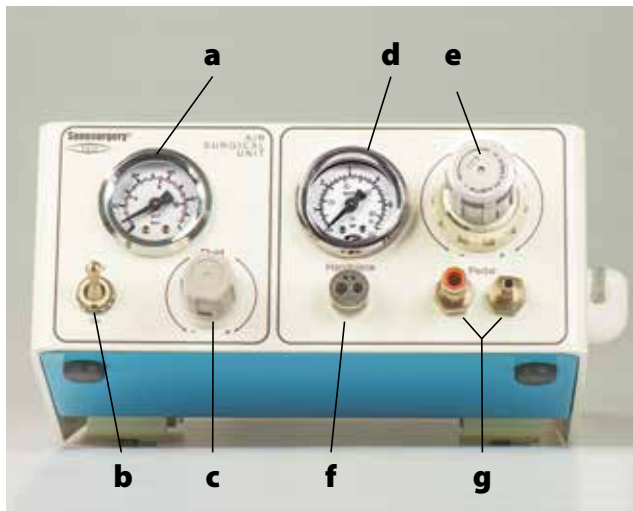
Fig. 1a

REAR PANEL CONNECTIONS (Fig.1b)

h) Quick male coupling (Camozzi 5000 Mini DN5 series) for the general inlet of compressed air (5 ÷ 8 bar).

i) Quick female coupling for pressure bag air (0 ÷ 0.6 bar)

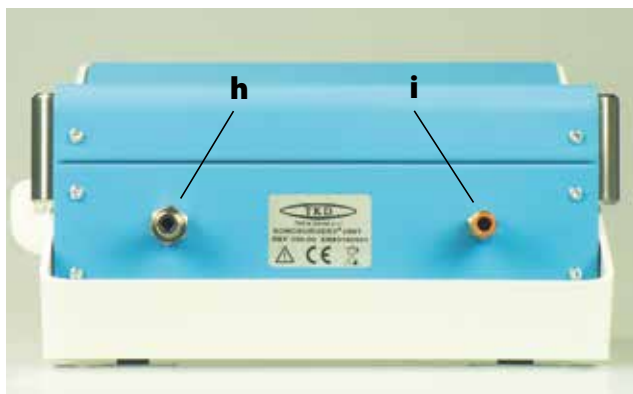
Fig. 1b

Good!

You have finished, now you can connect the handpiece and you are ready to use the surgical Unit.

TKD wishes you good work

ACTIVATION OF HANDPIECE AND STERILE IRRIGATION

The handpiece is started by pressing the foot on the control pedal. In order to avoid a sudden start and the consequent potential fracture of the inserts, it is recommended to start the thrust on the pedal gently and to reach maximum power by gradually increasing it.

With the switch (b) in the "AUTO" position, the pedal activates both the handpiece start-up and the sterile liquid flow automatically.

Note: At the end of each use the irrigation set roller clamp must be closed before removing the supply of compressed air to the Unit, so as to avoid unwanted liquid leaks.

CONNECTION, USE AND STERILIZATION OF HANDPIECE AND COUPLING

For connection, use and maintenance of the SONOSURGERY® AIR POWER handpiece and SONOSURGERY® STERIL rapid coupling, please refer to the corresponding operating instructions.

UNIT CLEANING AND DISINFECTION

Protect the Unit from direct sunlight and dust.

Cleaning of control Unit: rub the surfaces with a clean cloth soaked in a ordinary detergent product or with a disinfecting disposable wipe recommended for surgical instruments.

Cleaning of the detachable silicone hose: use a disinfectant based on alcohols like ethanol, methanol or a product containing glutaraldehyde up to 2%.

Do not use disinfectants containing benzalkonium chloride or acetone.

Do not immerse in a disinfectant solution or in a ultrasonic bath.

Note: The handpiece silicone cord cannot be sterilized in autoclave.

GUARANTEE

The Unit is guaranteed for a 24-month guarantee against all defects of material or faulty construction. Guarantee validity is subject to specific conditions.

TECHNICAL ASSISTANCE

Never disassemble the device. In case of malfunction or failure of the enclosure, and for any overhaul and repair work, please contact your usual supplier.

TECHNICAL SPECIFICATIONS

Max. pressure of input compressed air:

Degree of protection against ingress of water:

Standard Conformity:

Sizes:

Weight without hoses:

Working temperature:

Working humidity:

Storage temperature:

Storage humidity:

REFUSE DISPOSAL AT THE END OF LIFE

This device needs to be recycled. The user must return the equipment to its dealer or establish direct contact with an approved body able to process and derive value from this type of equipment (European Directive 2002/96/EC).

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